

Viewer Toolbar Visual Guide

Activate image to manipulate – Click left mouse button. GREEN band will show.



Close Study Icon - Closes current case study.



Layout Button - Allows user to change layout of study, e.g. 2x3 to 2x2.



Revert Layout Button - Allows user to revert to the most recently used layout.



Cine Tool - Activates a small cine toolbar at the bottom of your screen. This toolbar gives the user control over the viewing of a multi-frame image.



Reset Button - Resets all image settings to their original size and contrast.



Show/Hide Text Annotations Button - Turns text annotations on and off.



Window/Level Button - Allows users to adjust the contrast of an image. A drop-down menu to the right of the icon provides preset windows.



Zoom Button - Allows user to increase or decrease the image size. Move the image into the center of the screen by holding down the left mouse button and dragging. Hold down the right mouse button and drag upwards to zoom in on an image and similarly, drag downwards to zoom out.



Magnification Tool - Allows user to magnify a portion of an image. Hold down the left mouse button and drag mouse over image to display magnification.



Inverted Magnification Tool - Allows user to magnify a portion of an image in inverted contrast. Hold down the left mouse button and drag mouse over image to display magnification.



Clear All Measurements Tool - Removes all unsaved measurements in the active display.



Line/Ruler Tool - Allows the drawing for linear measurements by holding down the right mouse button and dragging. A line will be added to the active window. Release the mouse button to finish drawing. This will allow you to measure in a straight line.



Ellipse Tool - Allows the drawing of an ellipse by holding down the right mouse button and dragging to desired size. Release the mouse button to finish drawing. This will allow you to measure in a circle or oval SID measurement.